

## Medical Aids Subsidy Scheme (MASS)

# Guidelines

## BD Bodyguard T Syringe Driver



### Cleaning

Clean device **after each patient use**. Clean device and accessories **when visibly soiled/contaminated**:

#### Device

- Turn off device.
- Check housing for damage.
- Wipe external pump surface with **70% isopropyl alcohol wipe**.

#### Lock Box (if applicable)

- Clean using alcohol sprays and wipes.

#### Carry Pouch (if applicable)

- Clean using wet wipes containing water or alcohol.
- If thorough cleaning required, machine laundry at 60°C (do not spin wash, bleach, heat dry or iron).

These instructions are to be used alongside local Infection Prevention and Control Policy, NOT to replace.

### Storage

When storing the device, even if only for a short period, make sure you've checked the following:

- The device is back in its original packaging box.
- The box is sitting up the right way.
- It's stored in a dry, room-temperature location.
- If storing longer than 60 days, ensure battery has been removed (as per this image).



### Transport

- Within hospital (not in use) → Keep in original packaging box.
- Between hospital and patient home (not in use) → Keep in original packaging box.
- When in use → Keep inside lock box, inside carry pouch (contact MASS if you require these).
- Only transport to patient home via clinician. Do not send unattended with patient.

If pump is exposed to harsh conditions, suspected of damage or broken, **remove it from service** and **contact MASS**. For full manufacturer's instructions, refer to *Directions For Use* booklet included in syringe driver box.

© State of Queensland (Queensland Health) 2026



[creativecommons.org/licenses/by/3.0/au](https://creativecommons.org/licenses/by/3.0/au)

**For more information contact:**

Medical Aids Subsidy Scheme

PO Box 281

Cannon Hill QLD 4170 Australia

tel (07) 3136 3696

[MASS-SpecialisedServices@health.qld.gov.au](mailto:MASS-SpecialisedServices@health.qld.gov.au)

[health.qld.gov.au/mass](https://health.qld.gov.au/mass)

Medical Aids Subsidy Scheme (MASS) - Palliative Care Syringe Driver Program (PCSDP)

Guidelines - BD Bodyguard T Syringe Driver

Last updated: 12 June 2026